Smart Fitness Scale Balance de fitness intelligente

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User Manual • Guide d'utilisation

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Table of Contents

Package Contents	5
Specifications	5
Safety Information	6
Function Diagram	8
Digital Display	9
VeSync App Setup	10
Before First Use	11
Using the Smart Scale	11
Using the App	14
Maintenance	15
Troubleshooting	18
Warranty Information	21
Customer Support	23

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Thank you for purchasing the Smart Fitness Scale by Etekcity.

If you have any questions or concerns, please reach out to our helpful Customer Support Team at **support@etekcity.com**.
We hope you enjoy your new smart fitness scale!

Package Contents

- 1 x Smart Fitness Scale
- 3 x 1.5V AAA Batteries (Pre-Installed)
- 1 x Quick Start Guide

Specifications

Weight Capacity	180 kg / 28 st / 400 lb
Increments	0.05 kg / 0.1 lb
Weight Units	kg / st / lb
Platform	Tempered Glass
Dimensions	26 x 26 x 2.1 cm / 10.2 x 10.2 x 0.8 in
Battery	3 x 1.5V AAA Batteries
Auto-Off	30 seconds

READ AND SAVE THESE INSTRUCTIONS

Safety Information

Please read and follow all instructions and safety guidelines in this manual.

Medical

- This scale should not be used to diagnose or treat any medical condition.
- Scale data should not be used as a substitute for medical advice from a healthcare professional.
- This scale uses a harmless electrical current to measure body fat. Consult a healthcare professional before using the smart scale if you use medical electronic equipment such as a pacemaker.

General Use

- Check the scale before using. **Do not** use the scale if it is damaged in any way.
- **Do not** strike or hit the scale. **Do not** use the scale if the glass is cracked or broken. Due to the nature of tempered glass, the glass will shatter if there is even a slight crack. If you see a crack,

contact **Customer Support** immediately (page 23).


- The glass platform is slippery when wet. **Always** make sure the platform and your feet are dry before stepping on for measurement.
- Place the scale on a hard, flat surface to avoid tipping.
- To weigh safely and accurately, stand with your feet shoulder-width apart while on the scale. **Do not** stand on the edge of the scale or jump on the scale.
- Avoid standing on the scale for a long period of time, or when not weighing yourself.
- **Do not** overload the scale. Maximum weight capacity is 180 kg / 28 st / 400 lb.
- Keep the scale in a cool area with low humidity. Keep the scale away from heat sources, such as ovens or heaters. Avoid temperature fluctuations. Store the scale in a horizontal position, not a vertical position.
- Handle with care.
- Not for commercial use. Household use **only**.

Batteries

- **Do not** mix new and old batteries.
- If the low battery indicator (“Lo”) appears on the display, replace the batteries.
- If the scale is not in use for a long period of time, remove the batteries to extend the lifetime of the scale.

Function Diagram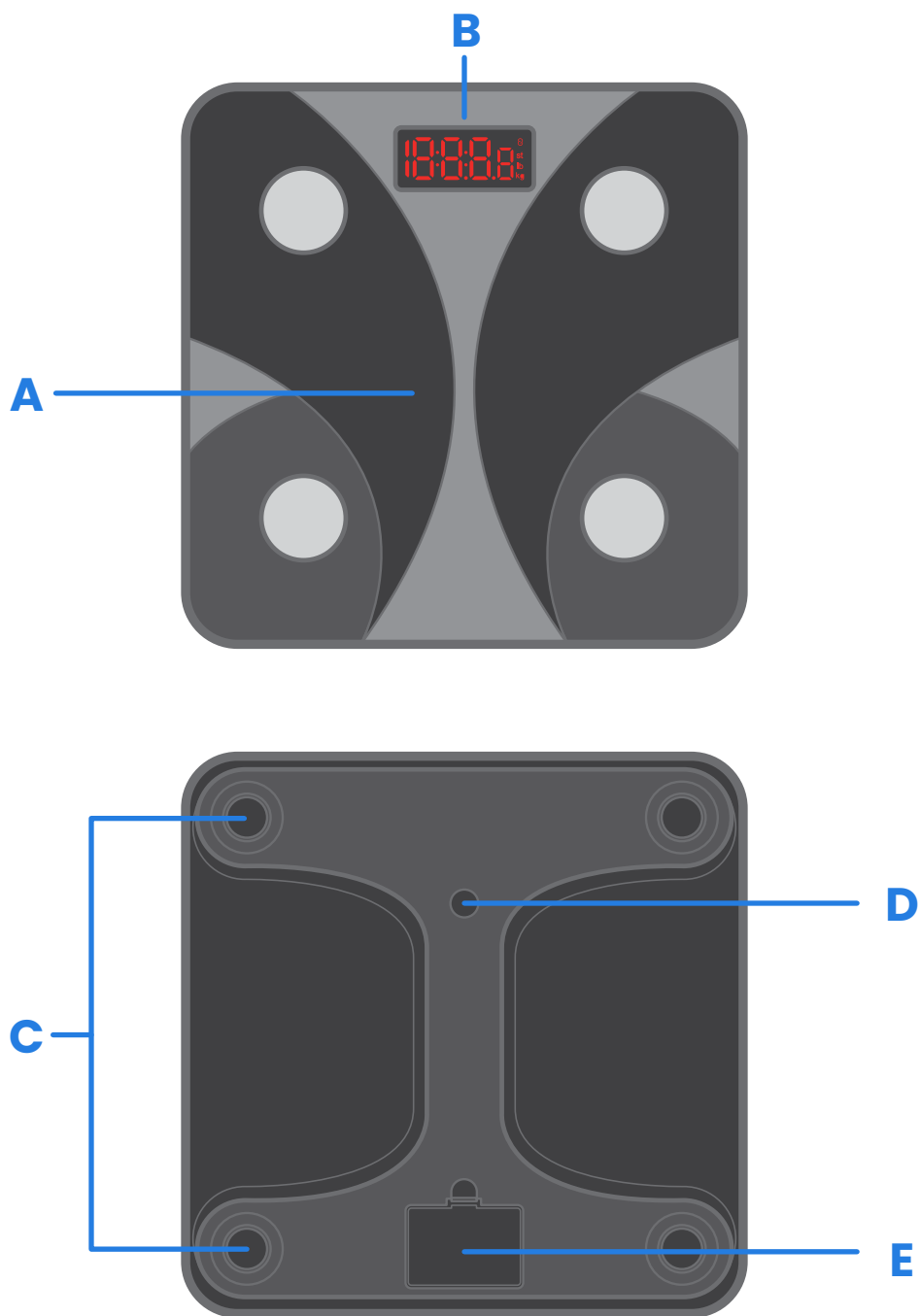

A. Weighing Platform

B. Display

C. Anti-Skid Padding


D. Unit Switching

Button

E. Battery Compartment

Digital Display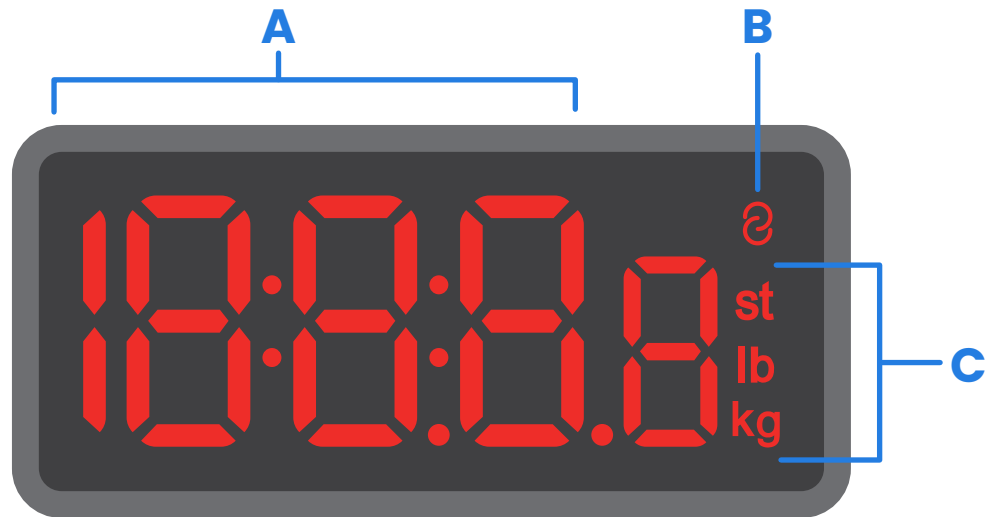

- A.** Weight Measurement
- B.** Wireless Connection Icon
- C.** Weight Units

VeSync App Setup

Note: Due to app updates and improvements, the contents of this manual may change without notice. The VeSync app user interface may appear slightly different.

1. To download the VeSync app, scan the QR code or search “VeSync” in the Apple App Store® or Google Play™ Store.

Note:

- This scale uses the VeSync app, not the VeSyncFit app.
- For Android™ users, you may need to select **Allow** to use VeSync.

2. Open the VeSync app. If you already have an account, tap **Log In**. To create a new account, tap **Sign Up**.

3. Make sure your phone has Bluetooth® turned on.
4. Tap **+** in the VeSync app and select your smart fitness scale.

Note: You can change the name and icon at any time in the smart fitness scale settings.

5. Follow the in-app instructions to connect to your smart fitness scale.

Before First Use

1. Open the battery compartment on the underside of the scale. Remove the included AAA batteries.
2. Remove the plastic insulator strip from the battery compartment.
3. Replace the batteries. Follow the guidelines in the battery compartment to match the + and - terminals.

Using the Smart Scale


Note:

- *If you are pregnant, scale measurements will not be accurate.*


Weighing Without the VeSync App

1. Place the scale on a hard, flat surface. [\[Figure 1.1\]](#)
2. Step on the scale, then step off. Wait for the screen to display "0.0" before using.
3. Place dry, bare feet on the scale's conductive areas. Stand on the scale until the display flashes several times, showing your final measurement. [\[Figure 1.2\]](#)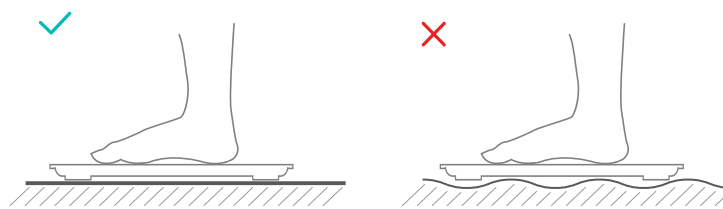

[\[Figure 1.1\]](#)

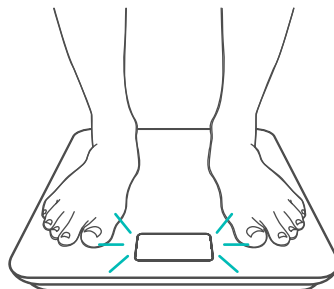
[\[Figure 1.2\]](#)

Switching Units

1. To change the unit of measurement (kg, lb, or st), step on the scale to turn it on. Then, turn the scale over and press the unit conversion button on the underside. To keep the settings consistent, make sure the scale is connected to the VeSync app. [\[Figure 1.3\]](#)
2. The default unit is pounds (lb). After changing units, the scale will show the new unit of measurement.

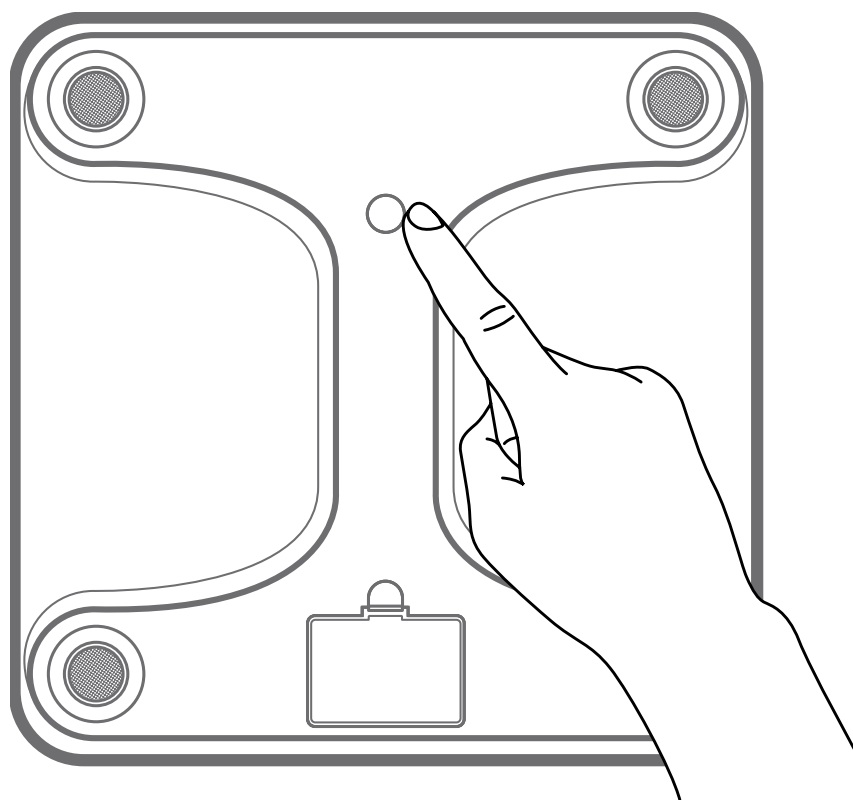[\[Figure 1.3\]](#)

Using the App

Creating a User

If you are a new user, you will need to enter your physical information before weighing yourself for the first time so the scale can calculate your body metrics.

Note:

- *Entering inaccurate information will make the scale's measurements less accurate.*
- *If you don't identify with the available gender options, or are unsure, pick the choice that you think best physically matches you. This choice is used for physical measurements, such as body fat percentages.*


Maintenance

Clearing the Scale

If the scale has been moved or flipped upside down, it must be cleared to ensure accurate results.

1. Place the scale on a hard, flat surface.
2. Step on the scale until digits appear on the display, then step off. [\[Figure 2.1\]](#)
3. The scale will show “0.00” when cleared. [\[Figure 2.2\]](#)

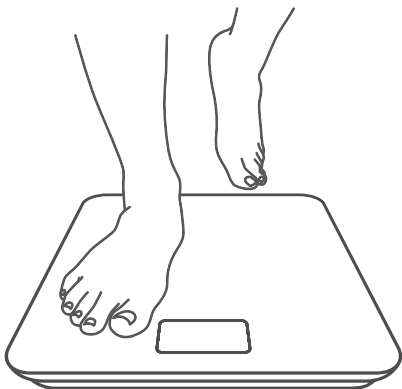[\[Figure 2.1\]](#)

[\[Figure 2.2\]](#)

Cleaning the Scale

- Use a slightly damp cloth to clean the surface of the scale, followed by a soft, dry cloth to wipe away any remaining moisture.
- **Do not** use chemical or abrasive cleaning agents to clean the scale.

Replacing the Batteries

The display will show “**Lo**” when the batteries need to be replaced.

1. Open the battery compartment on the underside of the scale.
2. Remove the old batteries and dispose of them properly.
3. Install 3 new 1.5V AAA batteries. Follow the guidelines in the battery compartment to match the **+** and **-** terminals.
4. After closing the battery compartment, clear the scale (see **Clearing the Scale**, page 15).

Display Messages

A 7-segment display showing the number '68.0'. The digits are formed by black segments on a white background. A small 'e' symbol is positioned to the right of the decimal point.

Wireless Connection

A 7-segment display showing the letters 'Lo'. The 'L' is formed by the left vertical bar, the bottom horizontal bar, and the bottom-right vertical bar. The 'o' is formed by the top horizontal bar, the right vertical bar, and the bottom horizontal bar.

Change Battery

A 7-segment display showing the letter 'E'. It is formed by the left vertical bar, the top horizontal bar, and the bottom horizontal bar.

Clearing

A 7-segment display showing the letters 'Err'. The 'E' is formed by the left vertical bar, the top horizontal bar, and the bottom horizontal bar. The first 'r' is formed by the right vertical bar, the top horizontal bar, and the bottom horizontal bar. The second 'r' is formed by the right vertical bar and the bottom horizontal bar.

Overload

Troubleshooting

The scale doesn't turn on.

- Check if the batteries are properly installed. If using the scale for the first time, remove the plastic insulator strip from the battery compartment.

Information flashes on the screen and then disappears.

- The batteries are low and may need to be replaced.

The smart scale can't be added to the VeSync app.

1. Please confirm the following things before pairing:

- The scale should only be connected to your phone via the VeSync App. Please do not connect the scale to your phone's Bluetooth® settings.
- Allow VeSync to access your Bluetooth® in your phone settings.
- Your scale and phone are within 30 ft (10 m) of each other.
- The scale is not currently connected to any other phone or app.
- Your phone's operating system is running on iOS® 12.0+ or Android 6.0+.
- You're using the latest version of the VeSync App.
- Turn off VPN (please ignore it if you are not using it).

Note:

- *For Android 12+ users, make sure that you turn on the nearby device permission and Bluetooth® to scan the target device. For users whose Android system is below Android 12, make sure that you turn on location and Bluetooth® to scan the target device.*

2. Try the following steps to pair your scale and your phone:
 - Enable Bluetooth® on your Phone.
 - Light up the scale and keep it on.
 - Open the Vesync App, and click “Add Device” or “+” on the top right corner of the home page. Choose “Health” to select your device model, and then follow the APP guides.

Note: Make sure to choose the right device model (you can find it on the back of the scale), or it will lead to a connection failure.

The data can't be synced to the VeSync app.

1. When the scale turns off, it will enter into power-saving mode and the scale and app will be disconnected. To sync data to the app properly, you need to reconnect the scale and the app through the following steps:
 - Restart the Bluetooth® of your phone.
 - Light up the scale and keep it on, then open the APP to enter the weighing homepage, wait for the Bluetooth® connection to be successful, then weigh on the scale.
2. Reset your scale or restart the Bluetooth® of your Phone:
 - Remove the batteries for 5-10 seconds, then put them back in.
 - Try restarting your phone's Bluetooth® after restarting the phone.
 - Try the methods in solution 1 again.
3. Bluetooth® detection:
 - Open the APP to enter the weighing homepage, click “Setting” on the top right corner, then “Troubleshooting”, and “Start Testing” to detect Bluetooth® anomalies.

My scale only measures my weight, not my body fat or other metrics.

- Set up your smart scale in the VeSync app. Your weight will show on the scale display, and more results will show on the app.
- Make sure Bluetooth® is turned on.
- If you're wearing shoes or socks, the scale cannot measure health metrics such as body fat. Step on the scale with bare feet.

- If your feet are too dry or too wet, the biometrics may not be detected. Since skin can become very dry during winter, please also try to apply some lotion or have a foot soak when it cannot measure other measurement values.

Note: *If the scale has errors on the display, will not turn off automatically, or has a problem not listed here, try removing the batteries for 3 seconds, then replacing them.*

If you still need help, please contact **Customer Support** (page 23).

IC Statement

This device contains licence-exempt transmitter(s)/receiver(s) that comply with Innovation, Science and Economic Development Canada's licence-exempt RSS(s). Operation is subject to the following two conditions:

1. This device may not cause interference.
2. This device must accept any interference, including interference that may cause undesired operation of the device.

Radiation Exposure Statement

This equipment complies with IC RSS - 102 radiation exposure limits set forth for an uncontrolled environment.

Warranty Information

Product	Smart Fitness Scale
Model	FIT 8S
<i>For your own reference, we strongly recommend that you record your order ID and date of purchase.</i>	
Order ID	
Date of Purchase	

TERMS & POLICY

Etekcitey Corporation (“Etekcitey”) warrants this product to the original purchaser to be free from defects in material and workmanship, under normal use and conditions, for a period of 2 years from the date of original purchase.

Etekcitey agrees, at our option during the warranty period, to repair any defect in material or workmanship or furnish an equal product in exchange without charge, subject to verification of the defect or malfunction and proof of the date of purchase.

There is no other express warranty. This warranty does not apply:

- If the product has been modified from its original condition;
- If the product has not been used in accordance with directions and instructions in the user manual;
- To damages or defects caused by accident, abuse, misuse or improper or inadequate maintenance;
- To damages or defects caused by service or repair of the product performed by an unauthorized service provider or by anyone other than Etekcitey;
- To damages or defects occurring during commercial use, rental use, or any use for which the product is not intended;
- To damages or defects exceeding the cost of the product.

Etekcitec will not be liable for indirect, incidental, or consequential damages in connection with the use of the product covered by this warranty.

This warranty extends only to the original consumer purchaser of the product and is not transferable to any subsequent owner of the product regardless of whether the product is transferred during the specified term of the warranty.

This warranty does not extend to products purchased from unauthorized sellers. Etekcitec's warranty extends only to products purchased from authorized sellers that are subject to Etekcitec's quality controls and have agreed to follow its quality controls.

All implied warranties are limited to the period of this limited warranty.

This warranty gives you specific legal rights and you may also have other rights which vary from state to state. Some states do not allow the exclusion or limitation of incidental or consequential damages, so the above limitation or exclusion may not apply to you.

If you discover that your product is defective within the specified warranty period, please contact Customer Support via support@etekcity.com. DO NOT dispose of your product before contacting us. Once our Customer Support Team has approved your request, please return the product with a copy of the invoice and order ID.

This warranty is made by:

Etekcitec Corporation
1202 N. Miller St., Suite A
Anaheim, CA 92806
USA

Customer Support

If you have any questions or concerns about your new product, please contact our helpful Customer Support Team.

Etekcite Corporation

1202 N. Miller St., Suite A
Anaheim, CA 92806
USA

Email: support@etekcity.com

Toll-Free: (855) 686-3835

Support Hours

Mon–Fri, 9:00 am–5:00 pm PST/PDT

**Please have your order invoice and order ID ready before contacting Customer Support.*

Attributions

Google, Android, and Google Play are trademarks of Google LLC.

App Store® is a trademark of Apple Inc.

iOS is a registered trademark of Cisco Systems, Inc. and/or its affiliates in the United States and certain other countries.

The Bluetooth® word mark and logos are registered trademarks owned by Bluetooth SIG, Inc. and any use of such marks by Etekcite Corporation is under license. Other trademarks and trade names are those of their respective owners.

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Etekcity Corporation
Google, Android et Google Play sont des marques de commerce de Google LLC.
App Store® d'Apple est une marque de commerce d'Apple Inc.
iOS est une marque déposée de Cisco Systems, Inc. et/ou de ses filiales aux États-Unis et dans d'autres pays.